

# HEBA-CON Lacquer (air-drying)

Distribution: HEBA-OTOPLASTIK Labortechnik u.  
Einrichtungen GmbH & Co. KG  
Hauptstr. 42 63853 Mömlingen  
☎ +49 6022 681600 ☎ + 49 6022 31663  
E-Mail: [info@heba.de](mailto:info@heba.de) [www.heba.de](http://www.heba.de)

## Application:

For all silicone based impressions & earmolds. Prior to application (dipping) the earmold must be absolutely clean, dry and free of grease! Hint: Use an appropriate surface cleaner (e.g. isopropyl alcohol). After shaking-off the surplus of lacquer, let the impression resp. the earmold (depending on coating thickness and humidity) dry for approx. 10 min at room temperature 23 °C / 73 °F. The surface of the impression/earmold shows now glossy layer. Apply lacquered otoplastics only when completely odorless.

## Important:

Clean bottle edge immediately after use, then close bottle tightly! To ensure the quality of HEBA-CON lacquer, do not return residues of lacquer back into original bottle. Do not store in the fridge.

## Please note:

During application of lacquer use a suction system!

## Safety advice:

On condition of a proper application, undesired effects are not to be expected. However, reactions of the immune system like allergies, irritations, cannot be absolutely excluded. In case of doubt, we recommend to make an allergy test before the application of the material.

For use by trained specialists.

Please refer to the safety data sheet!

Caution: Federal U.S. law restricts this device to sale by or on the order of a licensed healthcare practitioner (or trained specialist personal).

<b>Units:</b>	<b>HEBA-CON Lacquer</b>	10 ml	Art.-No. 52400
		100 ml	Art.-No. 52402
		250 ml	Art.-No. 52401

## Special storage instructions:

Keep tightly closed and store between 15 and 25 ° C.

**CE 0483**